

NEW LEMON
FRAGRANCE



AERIS DEFENCE™

HOSPITAL GRADE BROAD-SPECTRUM
ANTIMICROBIAL SURFACE DISINFECTANT

PROVEN EFFICACY AGAINST COVID-19 FOR UP TO 24 HOURS¹



HOSPITAL GRADE



**ACTIVE FOR
UP TO 24 HOURS**



**BIODEGRADABLE
RESIDUAL BARRIER**



**SINGLE STEP
APPLICATION**

BROAD-SPECTRUM ANTIMICROBIAL DISINFECTANT CLEANER WITH OUTSTANDING ONGOING PROTECTION

Aeris Defence provides long-term infection control with an invisible protective film, ensuring clinical surfaces are cleaner for longer.

- ✓ Kills COVID-19, influenza virus and a broad spectrum of bacteria
- ✓ Single step clean and disinfect after one application (no need to pre-clean surfaces)
- ✓ Provides on-going disinfection against COVID-19 for 24 hours with 9 ½ minutes contact time after a single application
- ✓ Compatible with most natural and synthetic surfaces susceptible to microbial colonisation

**24 HOUR
DISINFECTION
AND MICROBIAL
PROTECTION**

1. ARTG Certificate No. 387667 - April 2022

Active ingredients:
0.5% w/w Quaternary Ammonium Compounds, Benzyl-C12-16 Alkyldimethyl, Chlorides.

APPROVED FOR USE IN:

✓ Dental and General Practitioner Surgeries, Laboratories, Pathology and Medical Imaging Clinics:

Cleaning and disinfecting hard surfaces including tables, benches, counter tops, stools, door handles, bathroom facilities, fume cabinets and others.

✓ Child Care and Aged Care:

Cleaning and disinfecting hard surfaces such as cribs, change tables, cots and bed frames, as well as tables, benches, counter tops, chair, trolleys, door handles, telephones, intercoms, bathroom facilities and other hard surfaces.

✓ Hospitals and Clinics:

Cleaning and disinfecting hard surfaces in rooms and common areas such as floors, walls, door handles, bathrooms, bed frames, trolleys etc.

DIRECTIONS FOR USE:

To clean and disinfect spray liberally Aeri Defence onto the surface and wipe soiling with a clean cloth and leave to dry.

CAUTION: For food contact surfaces and items such as utensils, cutting boards, cookware: Rinse thorough with potable water before use.

NOTE: Aeri is not approved for disinfection of medical devices



CLEANS



PROTECTS



OPTIMISES



TO DISINFECT AND PROTECT FOR 24 HOURS

Spray on, wipe and allow to dry for 9 ½ minutes

TO ORDER:

Product Code	Barcode	Description	Dilution Rate
AG-AD-750	9352910002409	Aeri Defence 750mL Trigger Spray	Ready to use
AG-AD-5	9352910002416	Aeri Defence 5L	Ready to use



Listed in the Australian Register of Therapeutic Goods – ARTG No. 387667.
Always read the label and follow the directions for use.

Issue Date: December 2022

Aeri Environmental Ltd.
5/26-34 Dunning Ave, Rosebery NSW, 2018
info@aeri.com.au
aeri.com.au



MKT-237-05